


## **Electronic Pre-positioning – Model 3050**

*Refer to Figure 6*

Your Biscayne 3050 Dental Chair is equipped with automatic electronic pre-positioning. This feature is designed to provide the operator with the convenience of pre-positioning the patient at the touch of a button. You can preset the base and backrest positions as easy as 1-2-3.

Your chair will arrive with factory set pre-positioning. This is so that it can be checked for all quality control aspects prior to packaging and shipping. To find what pre-positioning the factory has set, simply depress and release the **Automatic Entry Button** located to the right of the foot control.

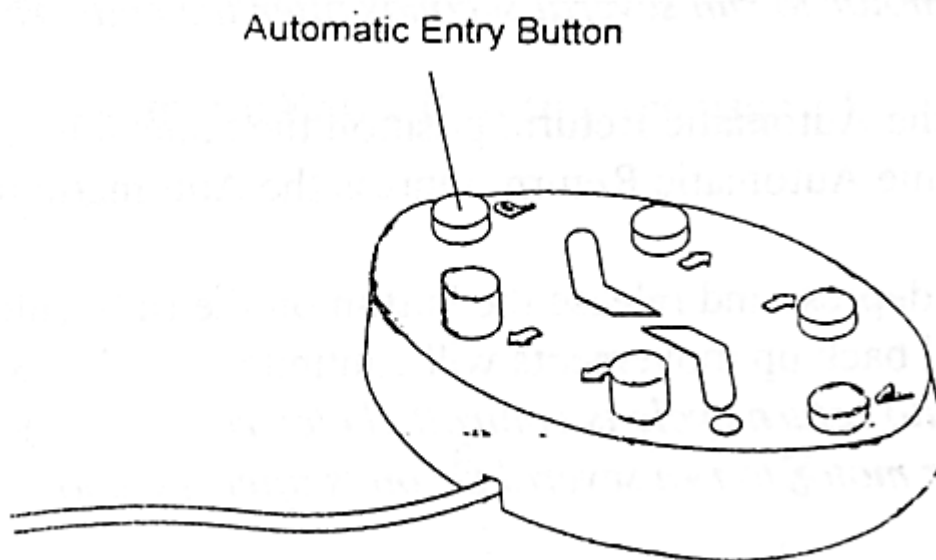

To set your personalized entry position proceed as follows:

1. Using the foot control, position the backrest and base of the chair in the desired position.
2. Depress and hold the **Pre-positioning Memory Button**, located on the upper right side of the cantilever (refer to figure 7, page 15), for 4 seconds. After releasing **the Pre-position Memory Button** allow 6 additional seconds for the new base and backrest positions to be stored in memory.
3. Your pre-positioning feature is now ready for use. To test the feature simply depress and release the Automatic Return Button (5). Allow the chair to reach the exit position. Depress and release the **Automatic Entry Button** (6). The chair will automatically return to your personalized entry position.

You can return the chair to your personalized position at any time by simply depressing and releasing the **Automatic Entry Button**. The personalized entry position may be reprogrammed as often as needed.

Should you want to interrupt the automatic movements of your Biscayne 3050 Dental Chair, press any manual function (Buttons 1-4) on the foot control. To resume the function, press the appropriate automatic function button.

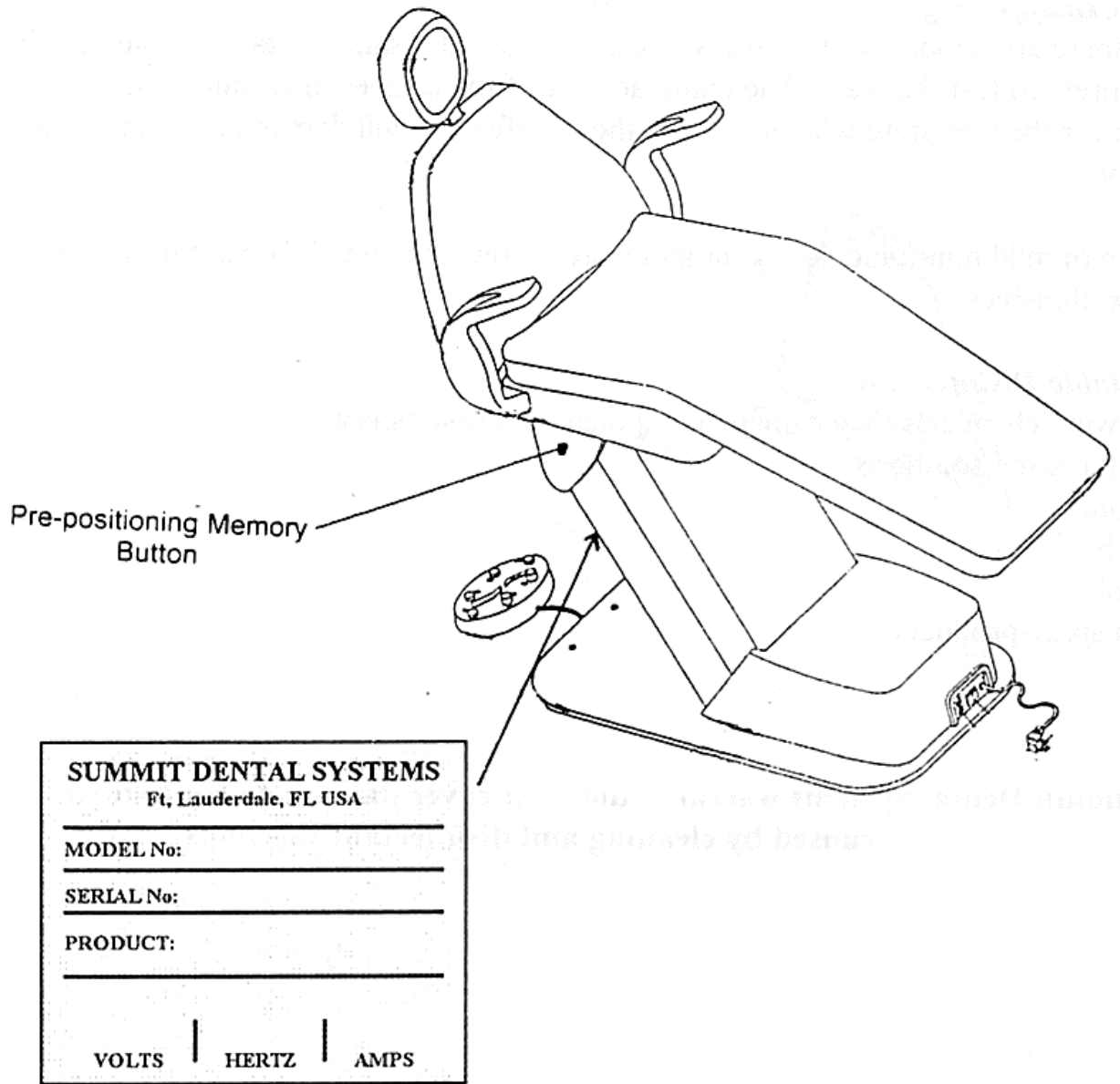


Figure 7